




RE-INSTALLING THE iOS myMERLIN™ PATIENT APP (APP1001)




Follow these steps to successfully undo the Bluetooth bond, uninstall and re-install the myMerlin app, and pair the patient's ICM to the myMerlin app.



1. Go to the phone settings and tap "Bluetooth."

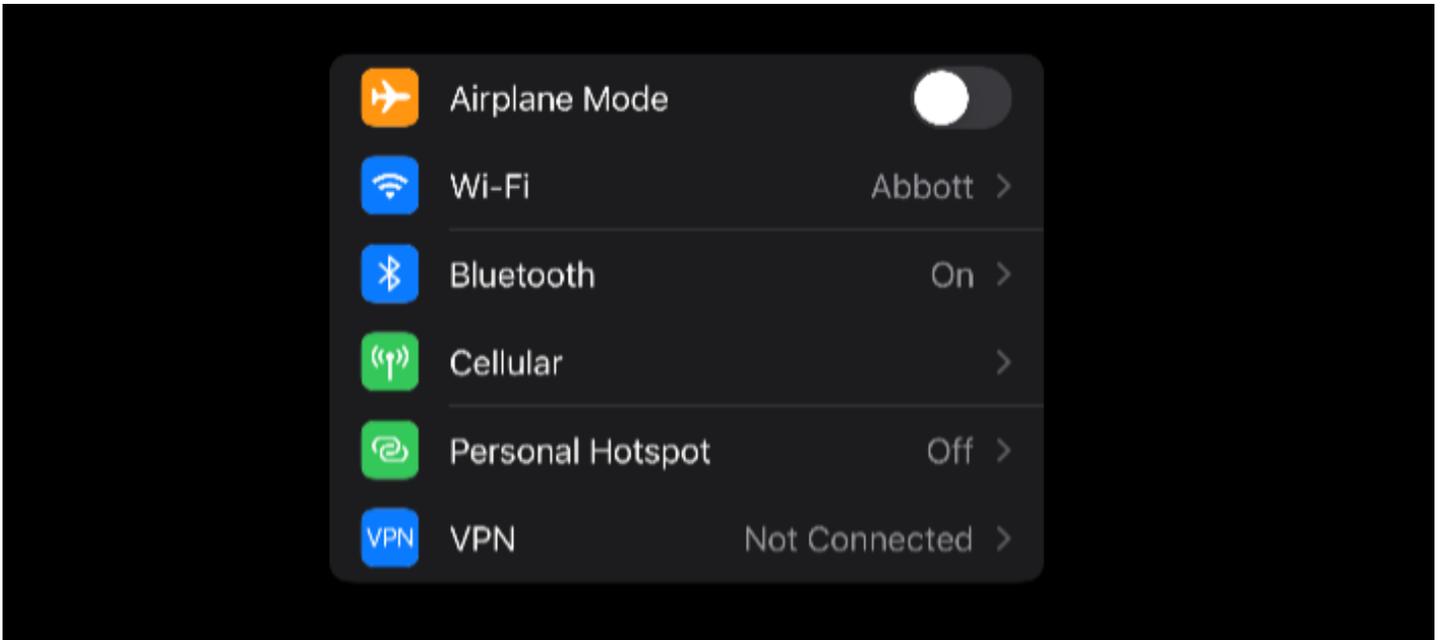




2. In the "Bluetooth" settings, there should be a device labeled as "DM" under "My Devices", tap the ⓘ icon to the right of "DM."

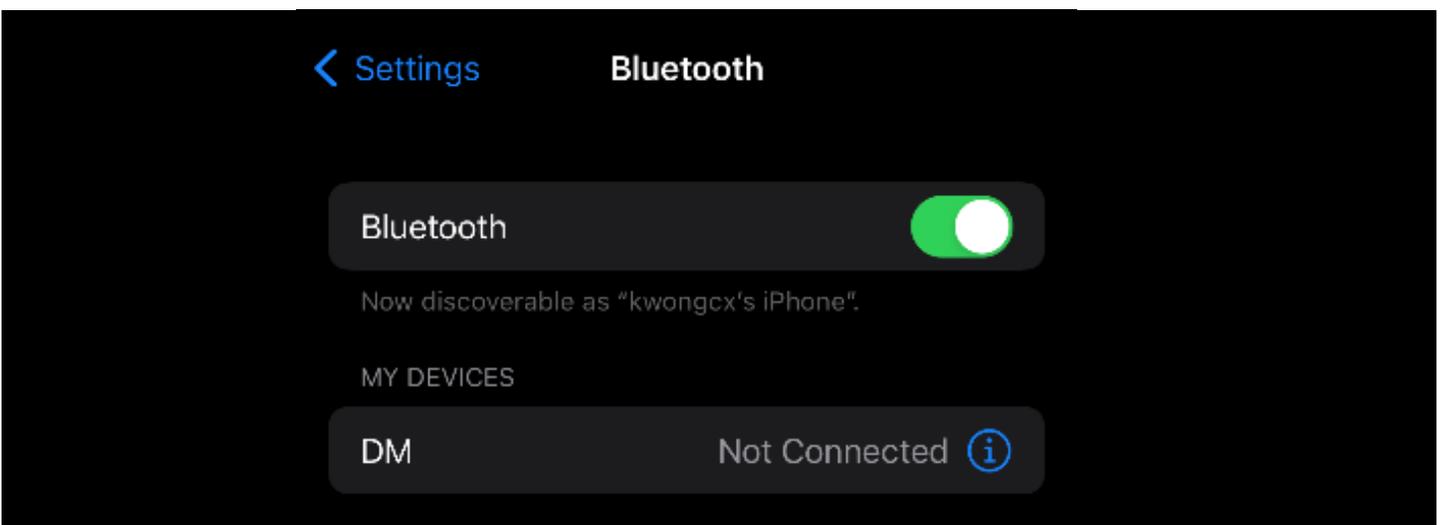





3. Tap “Forget This Device” and then “Forget Device.”

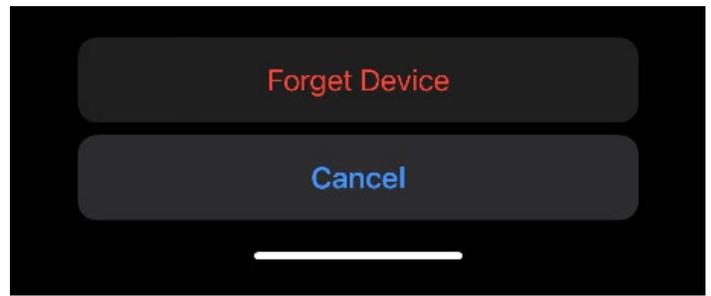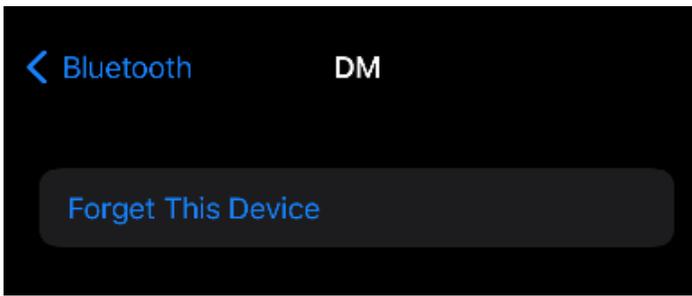


4. Proceed to uninstall the myMerlin app by holding your finger on the myMerlin app tile until a menu appears. Select “Remove App,” then “Delete App,” and then “Delete.”

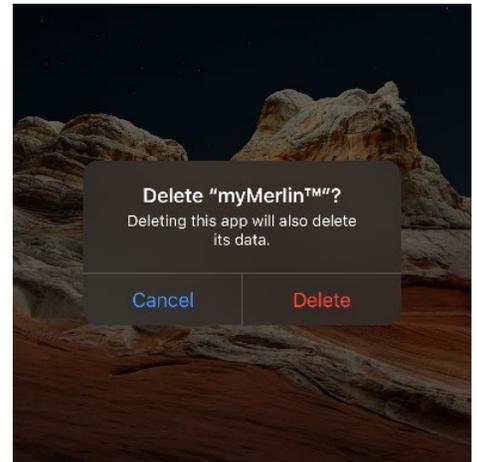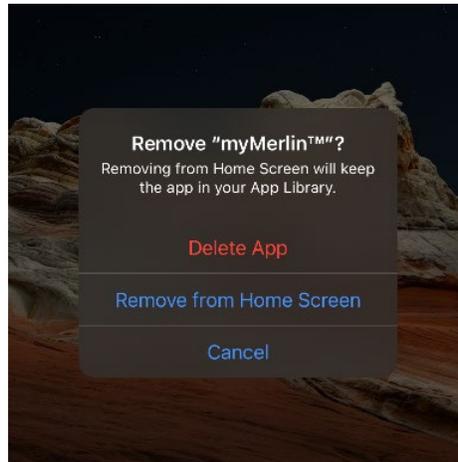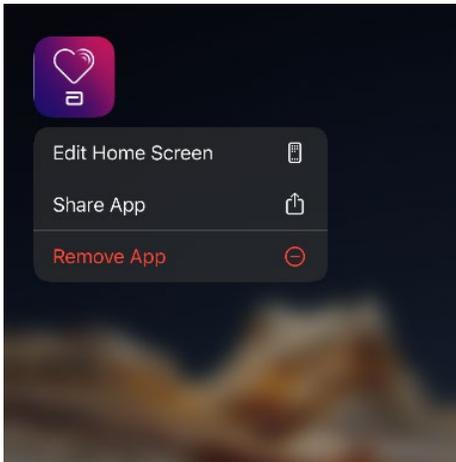


5. Proceed to the Apple App store and search for myMerlin. Tap the Cloud icon to begin reinstallation of the myMerlin app.

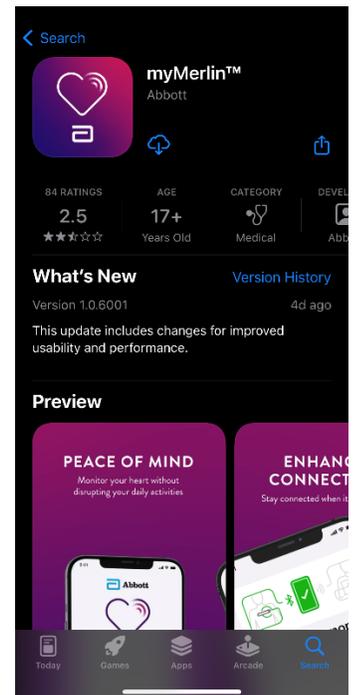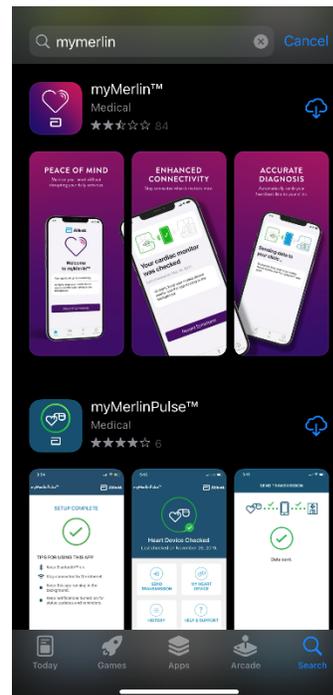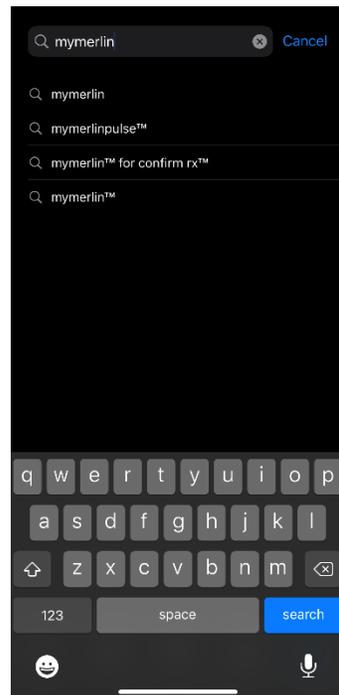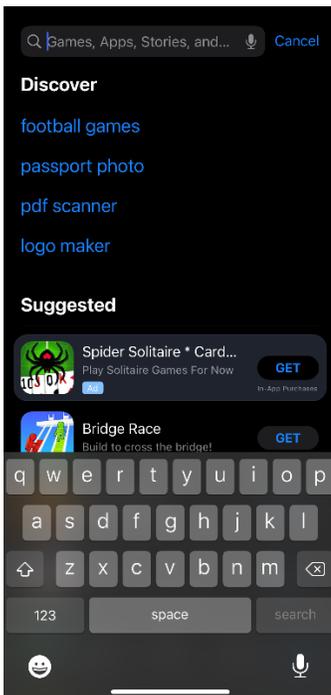


6. When the app has finished installing, open the app, select “OK” when prompted for Bluetooth access, and “Allow” for notifications from the myMerlin app.

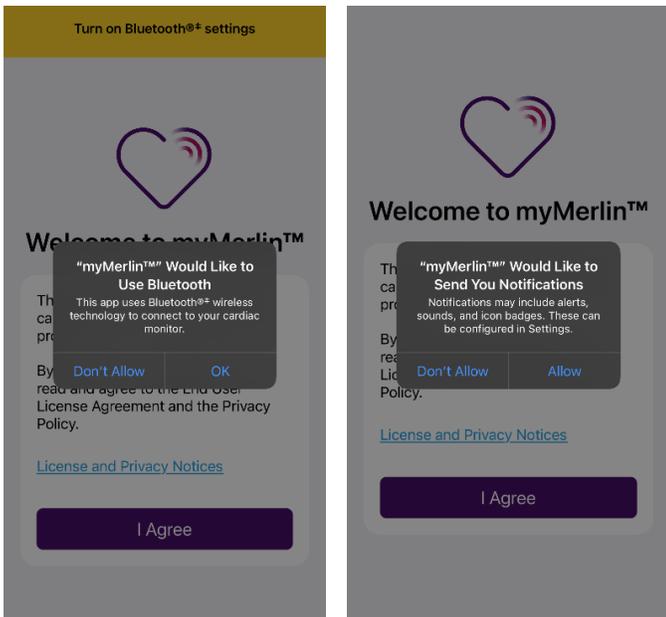


7. Select “I Agree” on the **WELCOME** page, then “Set Up Now” on the **LET’S GET STARTED** page.

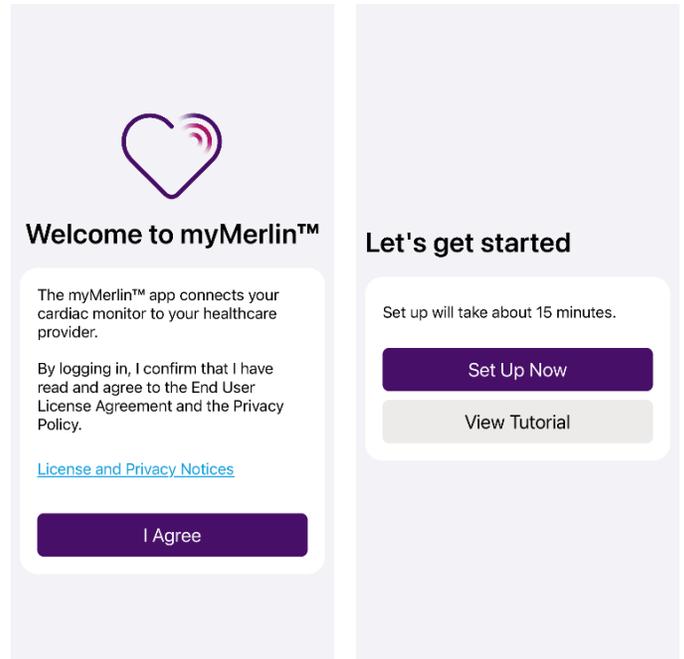


8. Enter the Date of Birth and Device Serial Number from the Patient ID Card.

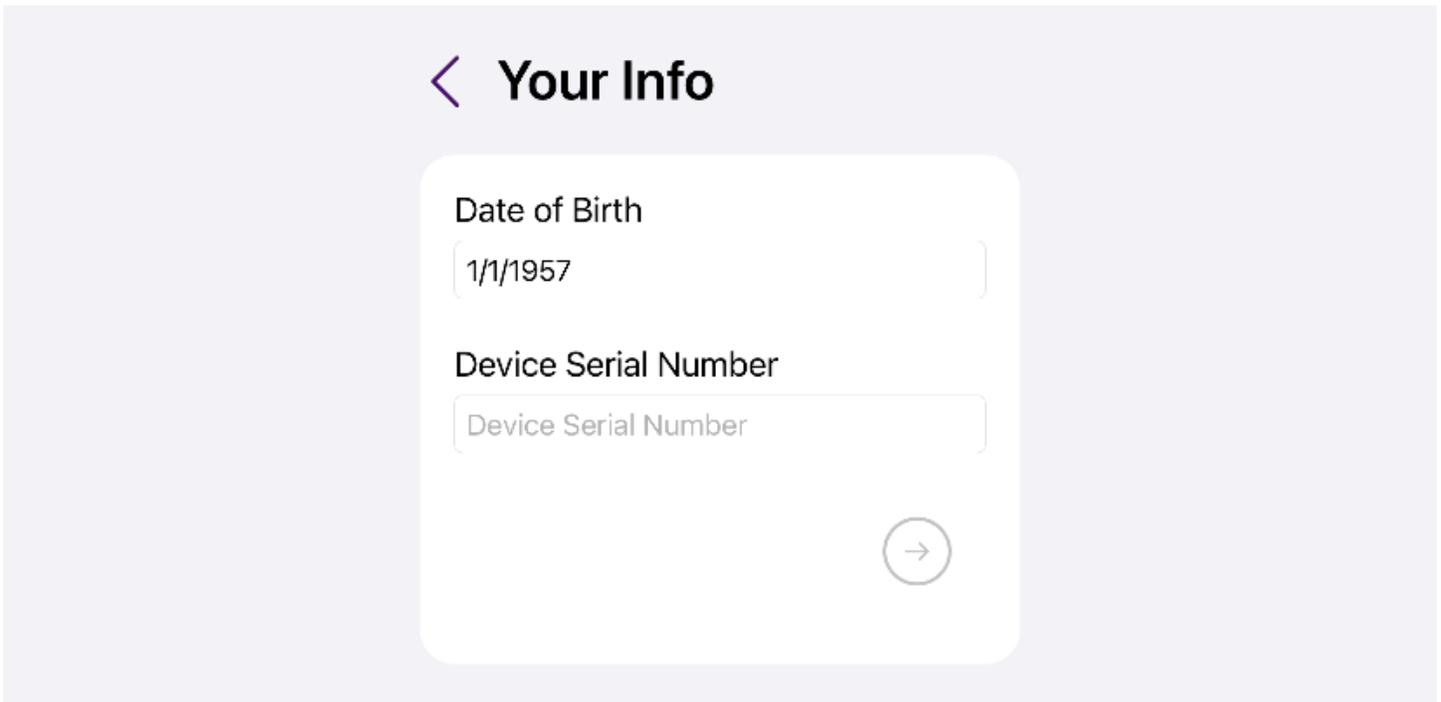


9. Select a method to receive the activation code (text message or email). Enter the code in the myMerlin app upon receipt.

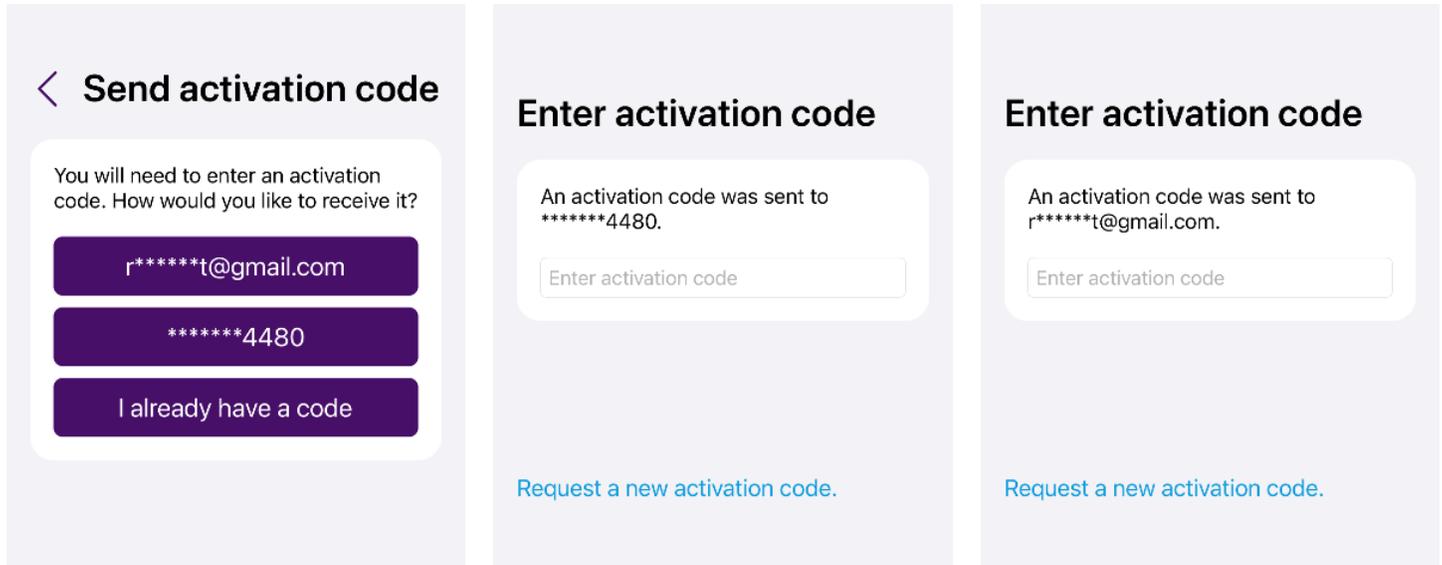

10. Select "Pair Now," and the pairing process will begin.

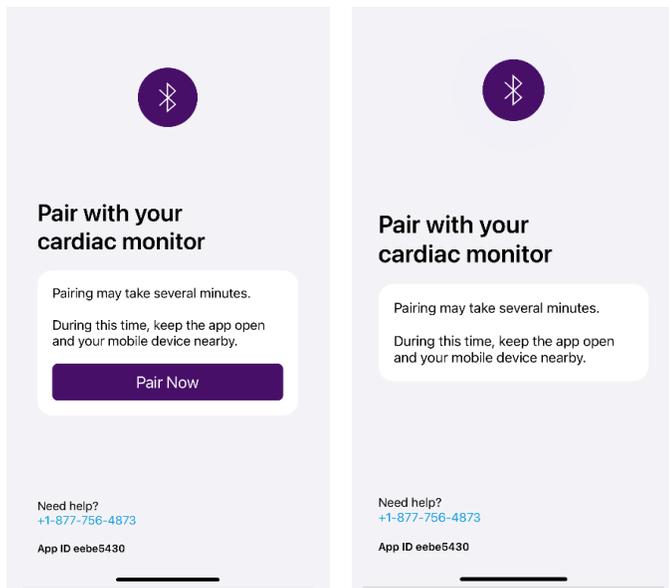

11. At the end of the pairing process, the **SET UP COMPLETE** page will be displayed.

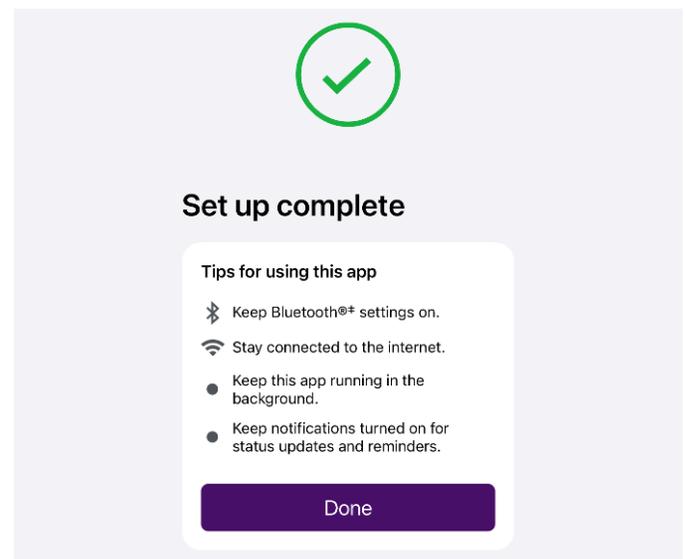

Abbott
15900 Valley View Court, Sylmar, CA 91342 USA
Tel: +1 818 362 6822
Cardiovascular.Abbott.com

Rx Only
Brief Summary: Prior to using these devices, please review the Instructions for Use for a complete listing of indications, contraindications, warnings, precautions, potential adverse events and directions for use.

™ Indicates a trademark of the Abbott group of companies.
‡ Indicates a third-party trademark, which is property of its respective owner.

This flyer is not intended to take the place of the manufacturer's patient manuals provided with each device.

© 2022 Abbott. All Rights Reserved.
MAT-2207185 v2.0 | Item approved for US use only.

Scan the
QR code to
learn more.

